CT-Gland-ProteX

REF 602251

Article No.: 999820V3

© 2019-03 SOMATEX Medical Technologies GmbH

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Manufactured by:
SOMATEX Medical Technologies GmbH
Hohenzollerndamm 150/151
14199 Berlin
GERMANY

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ENGLISH – INSTRUCTIONS FOR USE

Intended use: *CT-Gland-ProteX* reduces ionising radiation in order to protect the patient's thyroid gland during CT examinations.

Lead equivalent value: 0.125 mm Pb / 100 kV

Warning:

- The product may only be applied by specialist staff.
- Check the use-by date.
- For hygiene reasons, only use once.
- Be aware that artefacts occur on body parts near the site of product use.
- In the case of a topogram being taken before the image is recorded, the product should only be used afterwards.
- The product has not been designed to protect medical staff.
- The product must not enter the mouth.
- The product is not suitable for use on children.

Note:

- Moisturising the skin can negatively affect adhesiveness and should be avoided before application.
- Take note of the size of the product.

Application: Affix the product directly over the patient's thyroid gland using the adhesive strips (see cover sheet).

Storage: Store in a dry place. Protect from heat and sunlight (5 – 30 °C / 41 – 86 °F).

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