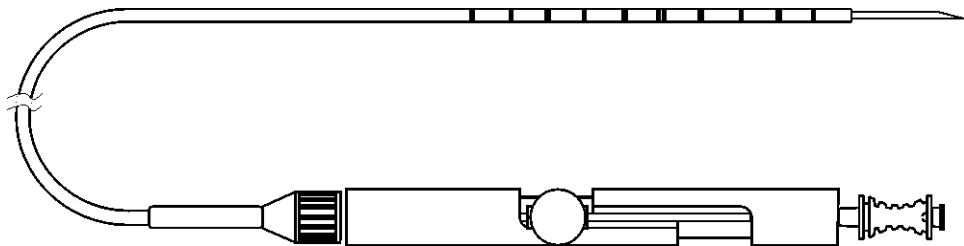
Broncho Cut Expert

REF

098307

098310

GEBRAUCHSANWEISUNG
INSTRUCTIONS FOR USE
MODE D'UTILISATION
ΟΔΗΓΙΕΣ ΧΡΗΣΕΩΣ
ISTRUZIONI PER L'USO
INSTRUÇÕES DE UTILIZAÇÃO
MODO DE EMPLEO
ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
NÁVOD K POUŽITÍ

INHALT / CONTENT

INHALT / CONTENT	2
DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ΕΛΛΗΝΙΚΑ	9
ITALIANO	11
PORTUGUÊS	13
ESPAÑOL	15
РУССКИЙ	17
ČESKY	19
SYMBOLE / SYMBOLS	21
INFO	23

STERILE	EO
---------	----

0482

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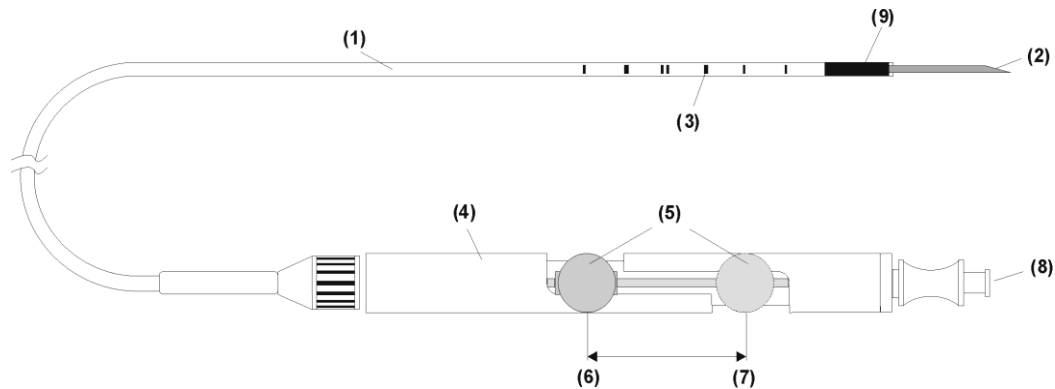
ENGLISH

Broncho Cut Expert

For transbroncheal needle aspiration (TBNA)

**Product description:**

SOMATEX® *Broncho Cut Expert* is used for bronchoscopically guided transbronchial needle aspiration (TBNA) for cytological diagnosis.

Contraindications:

In particular, the following contraindications apply for the use of the SOMATEX® *Broncho Cut Expert*: suspected tracheal process, haemorrhagic diasthesis or anticoagulation, pulmonary hypertension, stubborn coughing, contralateral pneumonectomy, poor compliance, presence of large bullous emphysema und echinococcal cyst.

Possible known complications:

Slight haemorrhage, pneumothorax

Notes:

The instrument is expressly intended for single use only. The product must be disposed of after use. The quality of the materials, coatings and adhesive bonds may deteriorate. Safe application is no longer guaranteed. The product is not designed for the cleaning and sterilisation processes that are necessary after a single use. The sterility of the reprocessed disposable products is therefore not guaranteed. The risk of undesirable injuries and infections, particularly cross-infections among patients and medical staff, increases disproportionately. SOMATEX® waives any responsibility in the event of resterilization and reuse of the instrument.

This product must only be used by qualified medical specialists.

Special information:

- During the introduction or withdrawal of the needle system into/from the bronchoscope it is essential that the puncture needle is retracted into the protective catheter (see also points 7 and 14). Otherwise, injuries and damage to the bronchoscope may occur.
- The needle must be removed from the region of the puncture if large quantities of blood accumulate in the guide catheter (1).
- A negative TBNA result cannot exclude a disease with certainty. Therefore, further examinations with alternative methods may be necessary.

It is essential to observe the following steps and information before use:

1. Before use, ensure that the package has not been opened and/or damaged; check the sterility date. Do not use if the package is damaged.
2. The needle set is suitable for use in combination with bronchoscopes featuring a working channel of 2.0 - 2.6 mm diameter. It is explicitly stated that the information and advice given in the instructions for use of the bronchoscope which is used must be observed.
3. Open the package.
4. Prepare a stop cock and a syringe (20 or 50 ml to create the vacuum); these are not included in the set.

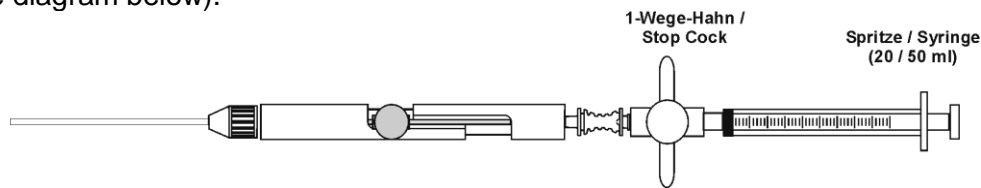
0482

Description of Application:

5. Remove the *Broncho Cut Expert* from the package under sterile conditions.
6. Connect the syringe to the Luer lock connection of the stop cock. Connect the free connection of the stop cock to the Luer lock connection (8) of the *Broncho Cut Expert* (see diagram below).

7. Before inserting the needle system into the bronchoscope, it is essential that a check of the system is performed with the following steps:
Extend the needle, which is retracted as delivered, by unlocking the slide (5). The needle now protrudes beyond the end of the catheter by 15 mm. Push the slide back to position (7) and lock it by turning. The tip of the needle (2) is now retracted into the puncture protection (9). If this is not the case, the product must not be used. Turn the stop cock to the "open" position.
 8. Introduce the extended bronchoscope into the trachea.
 9. After placing the bronchoscope, carefully insert the needle system into the working channel until the guide catheter (1) emerges from the channel.
 10. The previously selected puncture location is visually located on the basis of endotracheal or endobronchial distance measurement.
 11. With the bronchoscope held straight, the puncture needle must be fully extended by actuating the slide (5) (slide position (6)) and fixed. Now further insert the needle system together with the bronchoscope until the needle (2) perforates the bronchial wall and penetrates sufficiently deeply into the lesion.
 12. Now create a vacuum by pulling on the plunger of the syringe and turn the stop cock to the "closed" position.
 13. Make about 20 stabbing movements within the the tissue by moving the needle (2) rapidly backwards and forwards.
- Notice:** The slide (5) must remain locked in the front position (6).
14. The actual puncture procedure has now been performed. Before withdrawing the system from the bronchial mucosa, the puncture needle must be retracted into the guide catheter by actuating the slide (slide position (7)) and locked.
 15. Withdraw the needle system from the bronchoscope and then withdraw the bronchoscope.
 16. Turn the stop cock to the "open" position to release the aspiration material. Extend the needle with the aid of the slide (5), lock it and spread the sample onto a microscope slide by pressing the plunger of the syringe.
 17. Dispose of all components of the catheter system properly in a suitable container.

Storage Instructions:

Store in a dark, dry location at a temperature of between 5 and 30 °C (41 - 86 °F).

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SOMATEX Medical Technologies GmbH