CT-Eye-ProteX

REF 602252

**GEBRAUCHSANWEISUNG
INSTRUCTIONS FOR USE
MANUEL D'UTILISATION
MODO DE EMPLEO**

INHALT / CONTENT

INHALT / CONTENT	2
DE – DEUTSCH	3
EN – ENGLISH.....	4
FR – FRANÇAIS	5
ES – ESPAÑOL	6
SYMBOLE / SYMBOLS / SYMBOLES / SÍMBOLOS:	7

Please find your language online:

www.somatex.com/en/instructionsforuse

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EN – ENGLISH**Read instructions before use****Keep for future reference****Important Information:**

Read this instruction manual thoroughly and be familiar with its contents prior to use. Failure to read the entire manual and to familiarize yourself with all instructions before using the product is unsafe and can result in life-threatening or severe injury to the patient or user and in damage or malfunction of the device.

Intended purpose:

CT-Eye-ProteX reduces ionising radiation in order to protect the patient's eyes during CT examinations.

Contraindications: The product is not intended for use except as indicated above.

Patient target group: Adults

Performance characteristics: Reduction of ionising radiation

Lead equivalent value: 0.125 mm Pb / 100 kV

Warning:

- The product should be used by healthcare professionals only.
- Check the use-by date.
- For hygiene reasons, only use once.
- Be aware that artefacts occur on body parts near the site of product use.
- In the case of a topogram being taken before the image is recorded, the product should only be used afterwards.
- The product has not been designed to protect medical staff.
- The product must not enter the mouth.
- The product is not suitable for use on children.

Note:

- Moisturising the skin can negatively affect adhesiveness and should be avoided before application.
- The product should be positioned over the eyes and not used in combination with protective glasses.
- Take note of the size of the product.

Application:

Affix the product directly over the patient's eyes that they are completely covered using the adhesive strips (see cover sheet).

Storage: Store in a dry place. Protect from heat and sunlight (5 – 30 °C / 41 – 86 °F).

Any serious events that occur in relation to the product should be reported to SOMATEX Medical Technologies GmbH as well as the competent national authority.

SOMATEX®
MEDICAL TECHNOLOGIES GMBH

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