CT-Gland-ProteX

REF 602251

GEBRAUCHSANWEISUNG
INSTRUCTIONS FOR USE
MANUEL D'UTILISATION
MODO DE EMPLEO

Article No.: 999820V4

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Keep for future reference

Important Information:

Read this instruction manual thoroughly and be familiar with its contents prior to use. Failure to read the entire manual and to familiarize yourself with all instructions before using the product is unsafe and can result in life-threatening or severe injury to the patient or user and in damage or malfunction of the device.

Intended purpose:

CT-Gland-ProteX reduces ionising radiation in order to protect the patient's thyroid gland during CT examinations.

Contraindications: The product is not intended for use except as indicated above.

Patient target group: Adults

Performance characteristics: Reduction of ionising radiation

Lead equivalent value: 0.125 mm Pb / 100 kV

Warning:

- The product may only be applied by healthcare professionals only.
- Check the use-by date.

- For hygiene reasons, only use once.
- Be aware that artefacts occur on body parts near the site of product use.
- In the case of a topogram being taken before the image is recorded, the product should only be used afterwards.
- The product has not been designed to protect medical staff.
- The product must not enter the mouth.
- The product is not suitable for use on children.

Note:

- Moisturising the skin can negatively affect adhesiveness and should be avoided before application.
- Take note of the size of the product.

Application:

Affix the product directly over the patient's thyroid gland using the adhesive strips (see cover sheet).

Storage: Store in a dry place. Protect from heat and sunlight (5 – 30 °C / 41 – 86 °F).

Any serious events that occur in relation to the product should be reported to SOMATEX Medical Technologies GmbH as well as the competent national authority.

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© 2021-05-17 SOMATEX Medical Technologies GmbH